

# Changing Batteries

Your Nurse Call and/or Evacuation triggers and combined trigger + sirens are comprised of three elements: a backboard, the device, and two rechargeable batteries. You'll need to insert the batteries when you receive the device, then monitor ongoing to make sure you recharge them when needed.

## When Do Batteries Need Changing?

If you're using devices without a TCU or smartphone app, then you'll need to listen to the coordinating trigger. If it starts to beep, this indicates the batteries on one or more of the devices need recharging. You'll need to manually check the batteries of every connected device.

If you're using a TCU or smartphone app, you can monitor the battery levels of each individual device. Three green bars is fully charged, white bars indicates the batteries should be recharged.



**Step 1:** remove the two Torx screws with the supplied Torx screwdriver. Gently detach the bottom of the grey case from the top.



**Step 2:** eject the battery cartridge and recharge the batteries in the supplied battery charger. Allow up to 3-4 hours charging time.



**Step 3:** replace the batteries into the battery cartridge as shown (both + ends facing up as shown).

**Step 4:** push the battery cartridge into the device (as shown on the right). The cartridge will click into place.



**Step 4:** replace the base plate of the case on the device and screw shut with the two Torx screws and Torx screwdriver.