

How To Raise A Nurse Call Alert

Call for medical assistance using the process below. NOTE: the sirens on these devices only relate to evacuations (and will sound during an evacuation event).



1) In a medical emergency press the **CALL** button in the middle

2) A **BLUE** light next to **SENT** indicates the alert is raised and a high pitched sound will ring for 3 seconds



3) Remain calm while you wait for personnel to respond

4) A **GREEN** light next to **RECEIVED** indicates someone has received the alert and is responding and a high pitched sound will ring for 3 seconds



5) If no one responds within 3 minutes, the **BLUE** light next to **SENT** will turn off - you will need to press the **CALL** button again

6) Cancel the alert at any time by pressing the **CANCEL** button